

# 多功能防护面罩

Multifunctional Protective face mask



多一层保护  
多一份安全  
More protection More safety



## 自产品系列

淡蓝色  
light blue

粉红色  
light purple

象牙白  
Ivory white

## 主要功能Main Function:

保护医生(口腔技工师)在手术、治疗、操作中不受到粉尘、飞沫等污物的侵害,达到安全防护的目的。该面罩结构合理,适合各种头型的医务工作者佩戴。

It is made of high quality plastic and PE material with high transparency. It can protect the dentist and dental technician from the dust and foam in the progress of operation and treatment. With the perfect adjustable design, it can be suited for every dentist.

## 使用方法Usage:

只要去除透明PC片的前后两层保护膜,按使用说明要求安装即可。可消毒、可反复使用。

Take out the membrane of the PC piece, then install according to the instruction. It can be disinfected and used for times.

## 包装Package:

每盒配两片防护膜,一个头罩。

There are 2 protective transparent piece and on head shelf in each package.

## 多功能防护面罩使用方法

Instruction of multifunctional protective face mask



1.按照示意图说明撕开保护膜,保护膜前后共有两层。  
Pull apart the protective membrane,there are 2 membranes in each side.



2.将已撕掉保护膜的透明片嵌入帽沿上的固定块。  
Embed the transparent piece which the membrane has been taken out into the head shelf.

3.将小的透明片按图要求装入。  
Embed the small transparent piece according to the picture.



上海宣宇医疗器械有限公司  
Shanghai Sheen Medical Instrument Co.,Ltd

地址:上海市番禺路381号慧顺科技园2楼

邮编:200052

电话:+86-21-52587399

传真:+86-21-52587388

网址:www.xuanyu-sh.com

电子邮箱:xymarketing@xuanyu-sh.com

ADD:2nd Floor,Huishun Science Area,No 381 FanyuRoad,

Shanghai PR China

P.C:200052

TEL:+86-21-52587399

FAX:+86-21-52587388

WEB:www.xuanyu-sh.com

E-mail:marketing@xuanyu-sh.com

各地代理商  
Agents around

上海宣宇医疗器械有限公司  
Shanghai Sheen Medical Instrument Co.,Ltd

电话: +86-21-52587399 网址: www.xuanyu-sh.com 更多详情请向我们索取详细资料